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Molar-Etch

Acid Etchant Gel (37% Phosphoric Acid) for enamel and dentin

Molar-Etch is a 37% phosphoric acid etchant gel, with optimal viscosity which allows excellent spreading of the gel on the tooth surface for maximum etching efficiency. While **Molar-Etch** does not drip even on a wet tooth surface, it is easily washed off and does not leave any traces.

Materials:

Phosphoric acid, thickener, pigment and deionized water.

Intended Use:

Molar-Etch is designed for:

- 1- Etching of enamel surface or dentine before the preparation of a composite or bonded amalgam filling.
- 2- Before the preparation of a sandwich composite filling with a glass ionomer cement.
- 3- Before adhesive fixation of an aesthetic facet, either ceramic or composite.
- 4- Before the fixation of an inlay or onlay, the root pin, a crown or a bridge with a composite cement tissue since exposure to this product may injure the pulp.

Precautions:

This product is intended to be used by qualified dental professionals on a general patient population.

- Use personal protective equipment when using and handling this product.
- Use a rubber dam to isolate the treated tooth and to protect the surrounding soft tissues.
- **Molar-Etch** dispensing tips are designed for single use only, to prevent cross contamination between patients.
- Never use excessive force on syringe plunger to dispense etchant which may lead to over dispensing of the gel.
- If resistance is felt, replace current dispensing tip with a new tip and check flow outside the oral cavity.
- Close the syringe cap tightly after each use to prevent the product from drying in the syringe.

Direction for use:

- 1- Use a rubber dam to isolate the oral cavity.
- 2- Securely attach the dispensing tip to the Molar-Etch syringe intraoral and check the flow extraorally.
- 3- Apply **Molar-Etch** Gel to enamel and dentin surface for 15 seconds.
- 4- Rinse thoroughly for 10 seconds with water or until etchant has been completely removed. Gently air dry for 5 seconds.
- 5- Proceed with application of a bonding agent per manufacturer's instructions.

Warnings:

Molar-Etch contain 37% phosphoric acid. Avoid contact with skin, eyes, and soft tissue.

- Wash thoroughly with water after contact.
- Get medical attention for eyes.
- Avoid product application in direct contact with pulp.

Storage:

Store the product at 15 to 25°C

The expiration date is printed on the product packaging. Do not use the product after its expiration date.

Contents:

	3 × 2.5 ml
	3 × 25G

Expiry date

Manufacturer

Batch number

Product code

Consult IFU

Caution: See instruction for use

Limits of temperature of use and storage

15-25°C

Irritant

Rx ONLY

Sale by or on the order of a physician

For more information: See www.molardent.ir

Tirooj Kimia Daroo Co.
Science and Technology Park of Kurdistan, Sanandaj, Iran
Call at (98) 9190105876
Info@molardent.ir

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