9.5% BUFFERED HYDROFLUORIC ACID PORCELAIN ETCHANT GEL

INTRODUCTION:

EtchF Gel is a buffered 9.5% hydrofluoric acid gel designed to improve the mechanical bond between porcelain and adhesive/cement by increasing the surface area of the porcelain. The gel is buffered to reduce acidity risks when used correctly. **EtchF Gel** microscopically roughens feldspathic porcelain or glass surfaces, providing a controlled and effective alternative to sandblasting. **Note:** The gel's viscosity allows precise application, but extreme care is required due to the hazardous nature of hydrofluoric acid.

INTENDED USE:

- Etching porcelain restorations (e.g., veneers, crowns, inlays, onlays) prior to cementation.
- Repair of porcelain restorations.

COMPOSITION:

Hydrofluoric Acid (9.5%), Buffering Agent(s), Deionized Water, and Thickener

DIRECTIONS FOR USE:

- Preparation:** Operator and assistant must wear protective eyewear and gloves. Ensure the work area is clean and well-ventilated.
- Application:** If the restoration has not been etched by the lab, remove the cap from the **EtchF Gel** syringe, attach one of the provided 25G disposable micro tips, and verify gel flow. Apply a thin, even layer of **EtchF Gel** to the dry porcelain surface:
 - Apply for 90 seconds for porcelain.
 - Apply for 15 seconds for lithium disilicate (e.g., e-max).
- Rinsing and Drying:** Rinse thoroughly with copious water using a strong water stream or air-water syringe. Air dry. The etched surface should appear dull and frosty. If a white/chalky residue remains, agitate with a moist micro-brush to remove salt and debris, then rinse and dry again.

PRECAUTIONS:

- Avoid contact with glazed porcelain to prevent microscopic roughening and loss of gloss.
- Use in a well-ventilated area.

WARNINGS:

- Hydrofluoric acid is highly aggressive and a severe irritant to eyes, skin, and tissues. Vapors may irritate respiratory passages; chronic exposure may cause damage.
- Operator and assistant must wear protective eyewear and gloves. For intraoral use, a rubber dam is mandatory to protect the patient.
- Avoid contact with enamel, dentin, or glazed porcelain surfaces, as it may cause injury or loss of gloss.
- In case of skin, mucosa, or eye contact, rinse immediately with copious water for several minutes (15 minutes for eyes) and seek medical attention if needed.
- Do not ingest. Verify gel flow through the dispensing tip before intraoral application and use only recommended tips.
- Dispose of single-use components to prevent cross-contamination.
- Avoid re-etching restorations previously etched by the dental lab, as over-etching may weaken porcelain or reduce bond strength.
- Not all ceramic systems are etchable; consult the lab or manufacturer for compatibility.
- Check individual component labels for expiration dates.

CONTRAINDICATIONS:

Do not use in patients with known allergies to hydrofluoric acid.

DISPOSAL:

Dispose of unused product and empty syringe in accordance with local regulations for medical waste.

PACKAGE CONTENTS:

EtchF Gel (9.5%) is supplied in a 5ml luer-lock syringe with 3 blue, 25G disposable micro tips and instructions for use.

STORAGE:

- Store in a cool, dry place (15–25°C), away from direct sunlight.
- Keep syringe cap tightly closed to prevent contamination.

SHELF LIFE:

3 Years from the date of manufacturing

For more information: Visit www.molardm.com

	Expiry date		Batch number		Product code
	Manufacturer		Caution: See instructions for use	Rx ONLY	Sale by or on the order of a physician
	Consult IFU		Limits of temperature of use and storage		Irritant

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