RESIN-REINFORCED ZINC OXIDE EUGENOL (ZOE) CEMENT

INTRODUCTION:

REZINK is a high-strength, fast-setting, resin-reinforced zinc oxide eugenol (ZOE) cement engineered as an intermediate restorative material for temporary and select permanent dental restorations. The incorporation of resin significantly enhances the cement’s compressive strength and improves its adhesion to tooth structure, ensuring superior retention and durability during interim treatment phases. **REZINK** is ideal for temporary cementation of crowns, bridges, and other restorations, permanent cementation in low-load-bearing scenarios, and as a robust base or lining material under restorations. Its advanced formulation delivers reliable performance, making **REZINK** a trusted choice for clinicians seeking an effective and versatile intermediate restorative solution. Resin-reinforced ZOE composition provides sedative like qualities on hypersensitive tooth pulp.

INTENDED USE:

- Temporary cementation of crowns, bridges, inlays, onlays, and other restorations.
- Permanent cementation in specific low-load-bearing restorations (consult clinical guidelines).
- Intermediate restorative material for bases or linings under permanent restorations.

COMPOSITION:

- Powder: Zinc oxide, biocompatible resin
- Liquid: Eugenol, accelerator.

DIRECTIONS FOR USE:

1. **Preparation:** Ensure all instruments (spatula, mixing pad, scoop) are clean and dry. Work in a well-ventilated area at room temperature.
2. **Dispensing:** Gently shake the powder container to ensure uniformity. Using the provided scoop, dispense exactly 3 full, level scoops of powder onto a clean mixing pad. **Important:** “Use a spatula to level the scoop for precise measurement. Do not overfill or underfill the scoop, as an incorrect powder-to-liquid ratio may significantly change setting time and compromise performance.” Dispense exactly 2 drops of liquid from the bottle next to the powder.
3. **Mixing:** Using a clean spatula, gradually incorporate the powder into the liquid. Mix thoroughly for 20–30 seconds until a smooth, homogeneous consistency is achieved. **Working time:** 1–2 minutes at room temperature.
4. **Application:** Apply the mixed cement to the restoration or tooth surface using an appropriate instrument. Seat the restoration promptly and ensure proper alignment, leveraging **REZINK’s** resin-enhanced adhesion for secure placement.
5. **Setting:** Full setting occurs within 5–7 minutes, depending on ambient conditions.

PRECAUTIONS:

Avoid contact with skin, eyes, or oral tissues. In case of contact, rinse thoroughly with water and seek medical advice if irritation persists. Use adequate ventilation to minimize inhalation of eugenol vapors. Ensure proper isolation of the operative field to prevent contamination during application. Verify patient history for eugenol sensitivity before use.

WARNINGS:

- For professional dental use only.
- Do not use if packaging is damaged or if the product appears discolored or altered.
- Keep out of reach of children.

CONTRAINDICATIONS:

Do not use in patients with known allergies to eugenol or zinc oxide.

DISPOSAL:

Dispose of unused product and empty containers in accordance with local regulations for medical waste.

PACKAGE CONTENTS:

- 1 Bottle of **REZINK** Powder (30 g)
- 1 Bottle of **REZINK** Liquid (10 g)
- 1 Measuring scoop
- Instructions for use

STORAGE:

- Store in a cool, dry place (15–25°C), away from direct sunlight.
- Keep containers tightly closed to prevent contamination or moisture exposure.

SHELF LIFE:

3 Years from the date of manufacturing

For more information: Visit www.molardm.com

	Expiry date		Batch number		Product code
	Manufacturer		Caution: See instructions for use	Rx ONLY	Sale by or on the order of a physician
	Consult IFU	 15-25°C	Limits of temperature of use and storage		Irritant

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