TEMPORARY FILLING MATERIAL

INTRODUCTION:

Sillock is a radiopaque, saliva-curing paste designed for short-term dental restorations with the following unique advantages:

- ✓ Superior sealing properties, preventing any food or liquids in the mouth from leaking into the cavity, unlike light-curing temporary filling materials that may shrink and compromise the seal after curing.
- ✓ Optimal consistency and adhesion for quick and easy application.
- ✓ Eugenol-free, no effect on curing of composites.
- ✓ Fluoride release capability.
- ✓ Dimensionally stable with no excessive and annoying expansion.
- ✓ Pleasant taste and smell for greater patient comfort without a bitter aftertaste.
- ✓ Radiopaque for clear visibility on dental radiographs.
- ✓ Remains intact for 2 weeks and is easily removed using a dental excavator or ultrasonic scaler without fracturing.

INTENDED USE:

- Temporary fillings of Class I and II cavities.
- Temporary sealing in endodontics.

COMPOSITION:

Zinc Oxide, Calcium Sulphate, Biocompatible Polymer, Natrium Fluoride and Mint Flavor

DIRECTIONS FOR USE:

- Preparation:** Clean the cavity thoroughly using standard dental procedures and ensure it is slightly moist (not dry) before application.
- Application:** a) Dispense an appropriate amount of **Sillock** paste from the container onto a mixing pad. b) Using a sterile dental spatula, collect the paste and apply it to the cavity. c) Press gently to adapt the material to the cavity walls, ensuring a tight seal. d) Smooth the surface to achieve proper occlusion.
- Setting:** Filling surface will harden in about 20 minutes by absorbing saliva. Recommend the patient to avoid pressure or abrasion for at least 1 hour.
- Removal:** **Sillock** can easily be removed from the cavity in large pieces with a drill or a scaler. Rinse off any residue from the cavity with a water spray.

PRECAUTIONS:

In the case of direct contact with the oral mucosa, rinsing with water is sufficient. In the case of contact with the eyes, rinse thoroughly with water (10 min) and consult an ophthalmologist.

WARNINGS:

- Only supplied to dentists and dental laboratories or upon their instructions.
- Keep out of the reach of children!
- Should not be used after expiry date.
- Seal the jar tightly after every application.

CONTRAINDICATIONS:

Long-lasting temporary filling of cavities in vital teeth.

DISPOSAL:

Dispose of unused product and empty container in accordance with local regulations for medical waste.

PACKAGE CONTENTS:

- 1 Jar of **Sillock** paste (40 g)
- Instructions for use

STORAGE:

- Store in a cool, dry place (15–25°C), away from direct sunlight.
- Keep the container tightly closed to prevent contamination or moisture exposure.

SHELF LIFE:

3 Years from the date of manufacturing.

For more information: Visit www.molardm.com

	Expiry date		Batch number		Product code
	Manufacturer		Caution: See instruction for use	Rx ONLY	Sale by or on the order of a physician
	Consult IFU	 15-25°C	Limits of temperature of use and storage		Keep dry

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